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Produkt „Made in Germany“

DIRECTIONS FOR USE FOR BMV® SILIKONTUBE STANDARD/FENESTRATED

BMV® Silikontube

Item No. standard: 104012

Item No. fenestrated: 104512

EXPLANATION OF THE IMAGE SYMBOLS

Packing unit

The directions for use are intended to inform the doctor, nursing staff and patients/users about correct handling.

Please read carefully before use.

PURPOSE

The BMV® Silikontuben are used to keep the tracheostoma open. They offer the possibility of combining with other medical products, making it possible, for instance, to provide the patient with pulmonary rehabilitation and the speech function. The BMV® Silikontuben are inserted through the tracheostoma into the trachea and fixed in place with a cannula strap.

APPLICATION

Using the BMV® Silikontube (see Fig. 1).

Cleaning the BMV® Silikontube (see Fig. 2).

The BMV® Silikontuben must only be worn in combination with a suitable cannula strap. The BMV® Silikontuben are a single-patient product. The BMV® Silikontuben consist of an outer cannula with the possibility of connecting a speech valve and a HME (heat and moisture exchanger). The use of the various products is described in detail in the respective directions for use.

CLEANING

The BMV® Silikontuben are to be cleaned once a day with water (max. 65 °C) with a corresponding cannula cleaning brush without the addition of chemical additives.

The cannula is then to be cleaned with a care and disinfecting solution (e.g. BMV® Pflege- und Desinfektionslösung) according to the directions for use. The BMV® Silikontuben are also suitable for autoclaving.

Please note: BMV® Silikontuben must not come into contact with any type of oil!

The warranty becomes void if the cleaning regulations are not observed.

SAFETY NOTICE

There is a risk of forming pressure necrosis in the trachea or on the tracheal mucous membrane if the cannula is not used and worn properly. There is a risk of infection with improper cleaning. The use of unsuitable cannula straps could cause the cannula to not sit correctly in the tracheostoma.

Regular, meticulous cleaning and care as well as handling according to the information in these directions for use largely minimise the described safety risks.

SERVICE LIFE, STORAGE

We recommend replacing BMV® Silikontuben after approx. 6 months, calculated from first use. BMV® Silikontube tracheal cannulas that display minor damage must be replaced immediately. Unused cannulas should be stored in a dry environment, protected from sunlight and/or heat.

DISPOSAL

The product must only be disposed of according to the applicable national regulations for waste products.

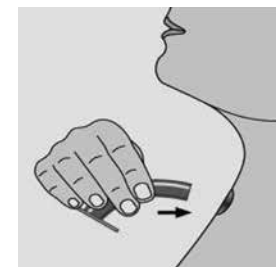
Fig. 1

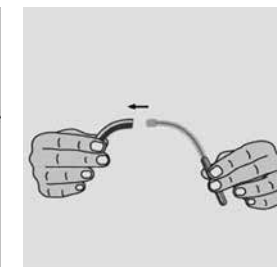
Fig. 2

BMV® Bender Medical Vertrieb GmbH
Gewerbegebiet Bitzen 23
53804 Much
Germany
info@bmv-med.de
www.bmv-med.de

Subject to product amendments
by the manufacturer at any time.

BMV® is a registered brand of BMV®
Bender Medical Vertrieb GmbH

Product made in Germany