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INSTRUCTIONS FOR USE BMV® WUNDTIEFENMESSER FLEXI

Wound depth gauge - Flexible wound depth measurement

Item number: 206080

The instructions for use are designed to provide information to doctors, caregivers and other medical professionals regarding correct use of the product.

Before using the product, please read these instructions carefully and keep them for future use.

Pictogram legend

	Manufacturer		Sterilized using ethylene oxide		Date of manufacture
	Catalog number		Do not resterilize		Use by
	Batch code		Single sterile barrier system		Keep away from sunlight
	Unique Device Identification		Do not reuse		Keep dry
	Medical device		Do not use if the packaging is damaged and follow the instructions for use		Content (in pieces)
CE 0197	CE marking with Notified Body				Consult instructions for use

Intended use

The BMV® Wundtiefenmesser Flexi is intended for use by medical professionals to measure the depth of wounds (as well as fistulas or cavity wounds).

Contraindications

There are currently no known contraindications for the BMV® Wundtiefenmesser Flexi.

Safety information

If the BMV® Wundtiefenmesser Flexi (or its sterile packaging) is damaged or its functionality is impaired, then it must be discarded!

The BMV® Wundtiefenmesser Flexi is sterilised with the ethylene oxide method. Sterility is only guaranteed if the protective packaging is not open, damaged or torn! The product is unable to be resterilised!

The BMV® Wundtiefenmesser Flexi is **ONLY** intended for single use! Cleaning and reprocessing are not permissible for hygienic reasons.

Insert **ONLY** the rounded end carefully into the wound; otherwise there is a risk of injury to the patient!

Description

The BMV® Wundtiefenmesser Flexi is a flexible gauge made of medical-grade plastic that can be manually formed and adapted to the wound. The rounded end ensures atraumatic measurement to the wound bed, as well as in the case of fistulas and cavity wounds. The BMV® Wundtiefenmesser Flexi is a 15 cm long gauge that provides for the measurement of depths in increments of 1 mm. The numbers 1 through 15 indicate measurements in centimetre increments. This enables the wound depth to be read out from the BMV® Wundtiefenmesser Flexi following insertion to the wound bed. The measurement is used for ongoing documentation in order to observe wound healing based on depth. **PLEASE NOTE:** In addition to wound depth, wound width, wound length and wound orientation should always be documented in relation to the corresponding body part. Please use a wound ruler for proper measurement of wound dimensions.

Duration of use

The BMV® Wundtiefenmesser Flexi is intended for single use only.

Application

Sterile disposable gloves must be worn during application. The procedure is as follows:

- Cleaning: Clean the wound properly and professionally per the general procedures for wound management or employer-specific procedures.
- Carefully open the sterile packaging and remove the BMV® Wundtiefenmesser Flexi.

3. Measurement: Apply gentle pressure to insert the rounded end of the BMV® Wundtiefenmesser Flexi vertically into the wound until the wound bed is reached. Read the measured position on the BMV® Wundtiefenmesser Flexi/at skin level. (Figure 1)

4. Remove the BMV® Wundtiefenmesser Flexi from the wound and make a note of the measured value.

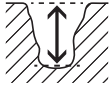
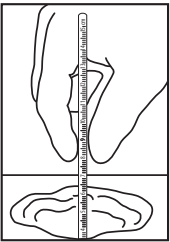
Figure 1: Wound depth measurement in cm with the BMV® Wundtiefenmesser Flexi

Storage

The product must be stored in dry conditions and protected from sunlight.

Disposal

The product must be disposed of only in accordance with the applicable national regulations for waste products.

Legal notices

In the event of any changes made to the product by the user or in the event of any use that does not fall within the scope of application described in these instructions for use, any warranty and liability claims against the manufacturer will be invalidated.

The manufacturer reserves the right to make changes to the product at any time.

All serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is resident.

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BMV® Bender Medical Vertrieb GmbH